R8212, R8222  
R8232, R8242  
R8312, R8322  
R8332, R8342

SELECT TL1 & TL2 toilet lifter – Operation and maintenance manual **en**

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

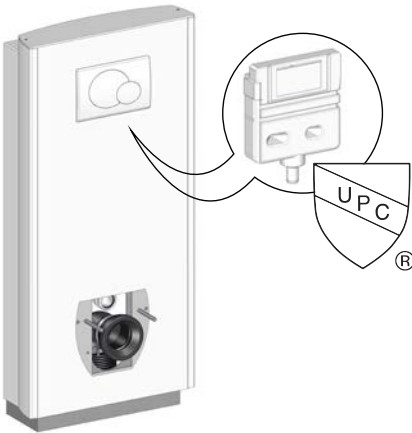
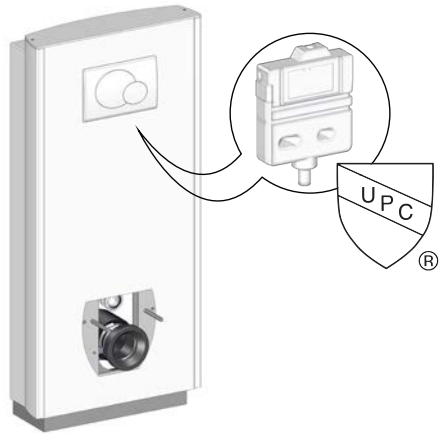
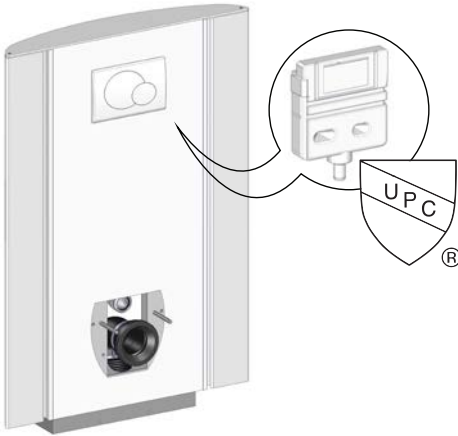
R8332

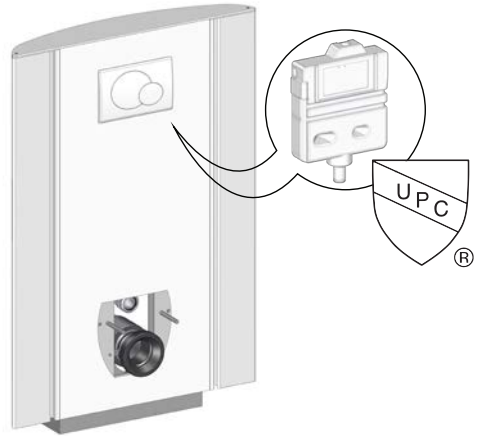R8342

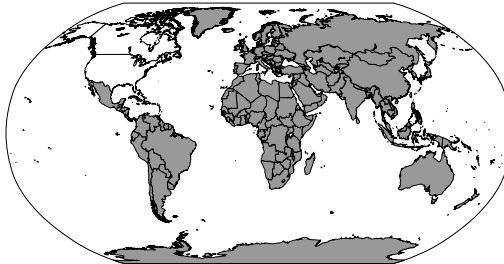A world map with shaded regions indicating 230V power standards. The shaded areas include Europe, Africa, Asia, and Australia.

A 230V power plug with two round pins.

230V

**REF** R8212, R8222  
R8312, R8322

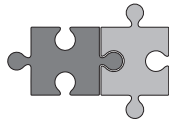
A world map with shaded regions indicating 120V power standards. The shaded areas include North America and parts of South America. An inset circle provides a magnified view of the North American region.

A 120V power plug with two flat pins.

120V

**REF** R8232, R8242  
R8332, R8342

R371370

R371470

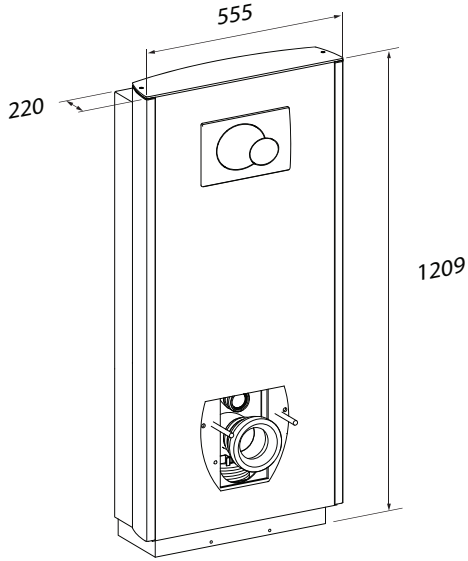
R371385


R371485


mm


R8212, R8222, R8232, R8242

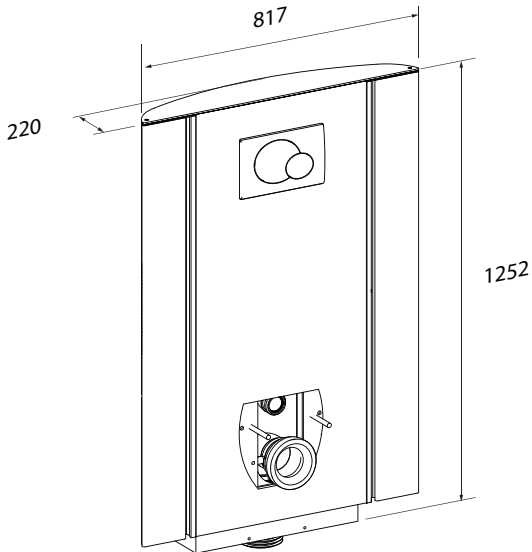R8312, R8322, R8332, R8342

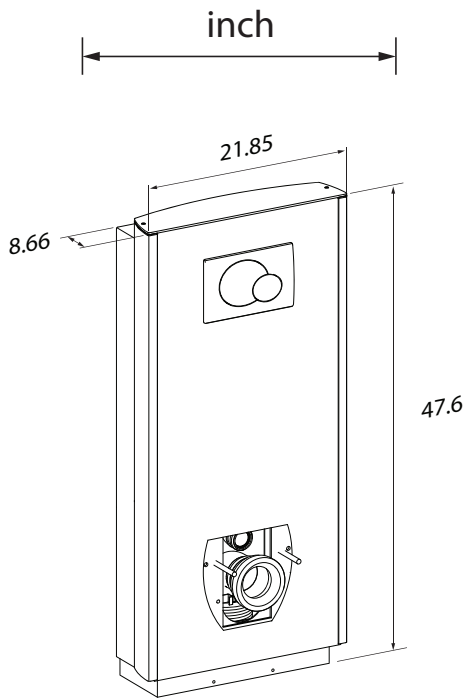R8212, R8222, R8232, R8242

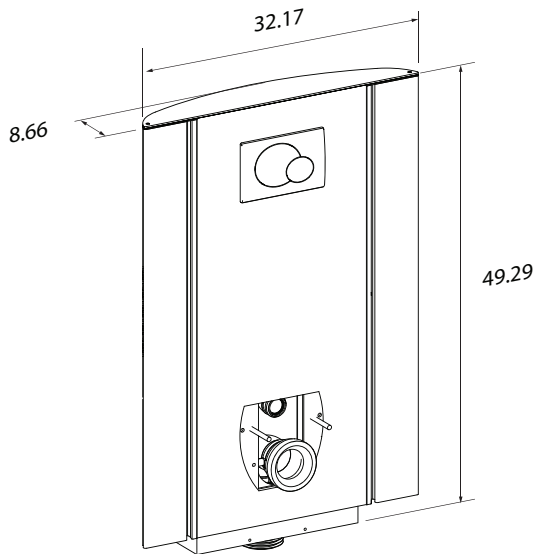R8312, R8322, R8332, R8342

# Content

Mounting instruction .....	9
Symbols used in this manual .....	10
General safety .....	11
Product label .....	12
Intended use .....	14
Intended operator profile .....	14
Operating environment .....	14
Operating instructions .....	15
Support arms (optional) .....	16
Cleaning .....	17
To remove limescale .....	17
Maintenance .....	18
Maintenance inspection .....	18
Technical data .....	20
Trouble-shooting .....	21
Disposal and recycling .....	21

## Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from [www.pressalit.com](http://www.pressalit.com).

# Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

**WARNING** indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

**CAUTION** indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

**IMPORTANT** indicates important information about handling and use of the product.

**NOTICE** is used to address practices not related to personal safety.

Hand injury warning

Head injury warning

Feet injury warning

Electricity or potentially dangerous voltage

Compatible products

# General safety

## **WARNING**

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than are specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

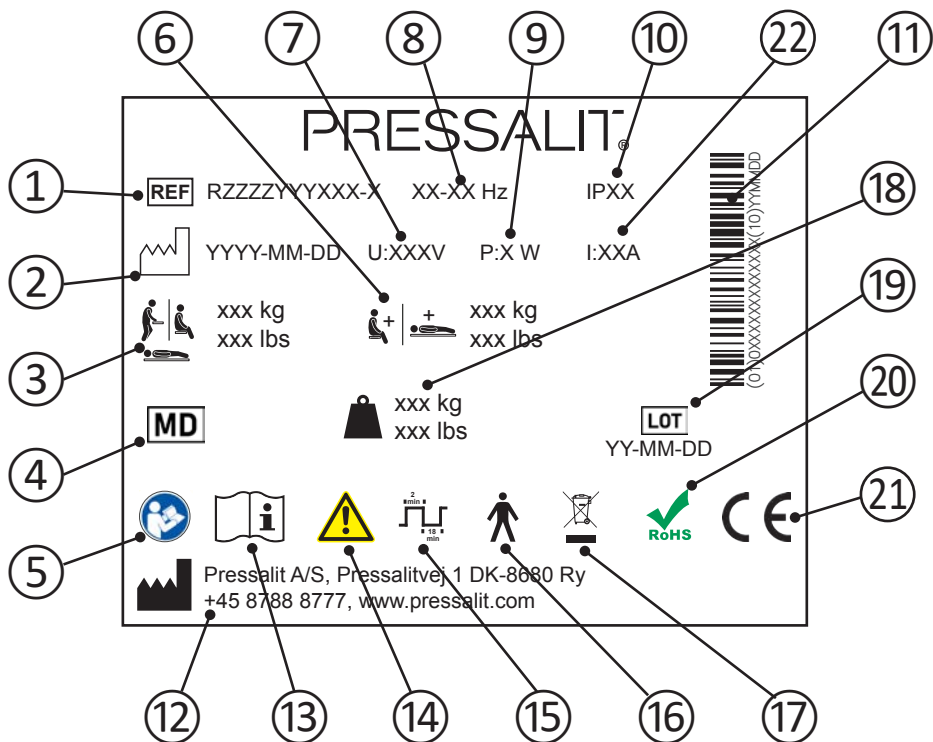
## **CAUTION**

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

## **NOTICE**

- The information in this manual is based on correct fitting of the product in accordance with our Assembly instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

# Product label

- ① Article number

---

- ② Production date

---

- ③ Maximum user weight

---

- ④ Medical device

---

- ⑤ Consult manual for use

---

- ⑥ Max load

---

- ⑦ Mains voltage

---

- ⑧ Frequency

---

- ⑨ Maximum power consumption

10	IP class
11	GS1-128 barcode
12	Name and address of manufacturer
13	Consult manual for use
14	Consult the manual for important cautionary information such as warnings and precautions
15	Duty cycle
16	Type B applied part Part of the product in normal use necessarily comes into physical contact with the patient to perform its function The product complies with requirements of IEC 60601-1 to provide protection against electric shock
17	Do not dispose as unsorted municipal waste. The product should be collected separately and returned to the designated recycling service.
18	Product weight
19	Batch code
20	This product does not contain any of the hazardous substances mentioned in the Council Directive 2011/65/EC on RoHS.
21	These products are CE-marked in accordance with relevant requirements in: - Regulation (EU) 2017/745 (5 April 2017) on medical devices. - Council Directive 2006/42/EC (of 17 May 2006) on machinery. - Council Directive 2011/65/EC (of 8 June 2011) on RoHS. A declaration of conformity can be found at <a href="http://www.pressalit.com">www.pressalit.com</a> .
22	Electric Current

Product label placement ①.

## Intended use

This product is intended for optimum ergonomics at a toilet, for both seated and standing users with disabilities.

## Intended operator profile

The product can be used by all - adults, children, young and old.

en

## Operating environment

For indoor use in a wet environment, such as a bathroom or wash room. The product is intended to be used at room temperature in private homes, nursing homes and hospitals under normal working conditions.

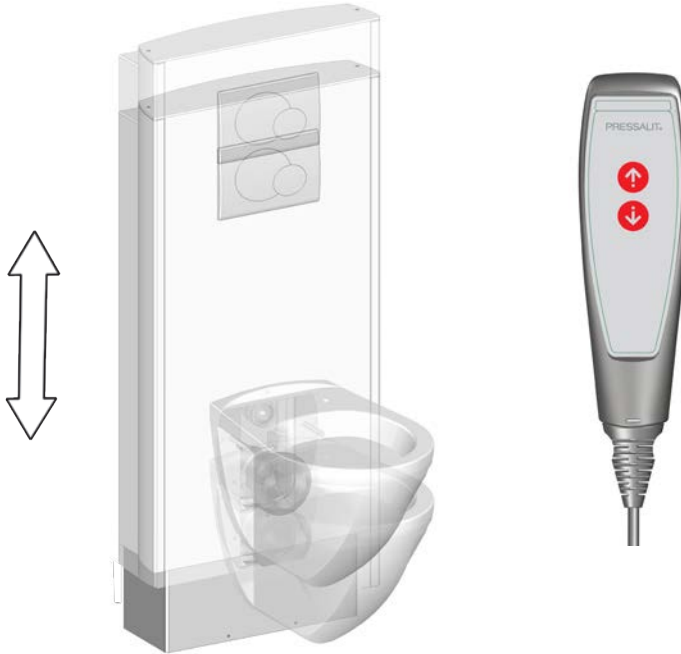
The surface of the product may become hot and / or cold if exposed to external sources of heat or cold (eg sunlight or frost).

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities.

# Operating instructions

The toilet lifter can be adjusted 400 mm / 15¾" up and down.

**Max user weight: 200 kg / 440 lbs**

**Max load: 235 kg / 518 lbs**

The height adjustment is controlled by the wired remote control unit.

Press and hold the Up button to raise the unit, and press and hold the Down button to lower the unit. When the button is released, the motion stops.

**Take care to avoid injury to hands, feet and head when in the vicinity of moving mechanical parts.**

**CAUTION**

Never place objects underneath the toilet.

# Support arms (optional)

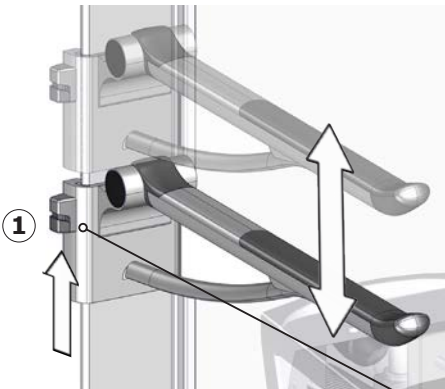
The toilet lifters model R8312, R8322, R8332 and R8342 can be equipped with support arms.

The support arms can be folded up and down. Fold them up when not in use.

The support arms can also be adjusted 240 mm / 8<sup>7</sup>/<sub>16</sub> inches in height.


en

To adjust upwards, pull the support arm upwards.

To adjust the support arm downwards, pinch the locking button ① and pull the support arm a little bit upwards while holding the button. Then move the support arm downwards. Release the locking button to secure the support arm in place.

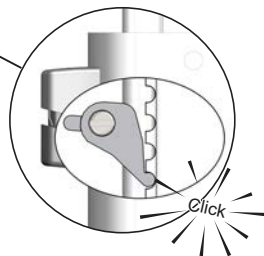Make sure that the locking button pin is engaged in the notch.

# Cleaning

The product is to be cleaned with hot water or a mild soapy cleaning agent. A list of suitable cleaning agents tested by Pressalit can be found on [www.pressalit.com](http://www.pressalit.com).

## **CAUTION**

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

## **IMPORTANT**

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids
- Never spray the product directly with a high pressure cleaner.
- Avoid use of abrasive, corrosive or chlorine-based cleaning or disinfecting products.

### **To remove limescale**

To remove limescale marks on the aluminium parts, use 3 parts 30% household vinegar to 7 parts water and afterwards wipe the parts with a cloth wrung out in clean water.

In countries with relatively high levels of lime in the water supply, it may be necessary to remove limescale deposits from the aluminium parts using a de-scaling agent such as acetic acid. Test this first on a area that is not visible when the product is in use.

# Maintenance

Expected service life of the toilet lifter is 10 years, provided that maintenance is carried out once a year as described below.

## Maintenance inspection

Take the toilet to its highest position.

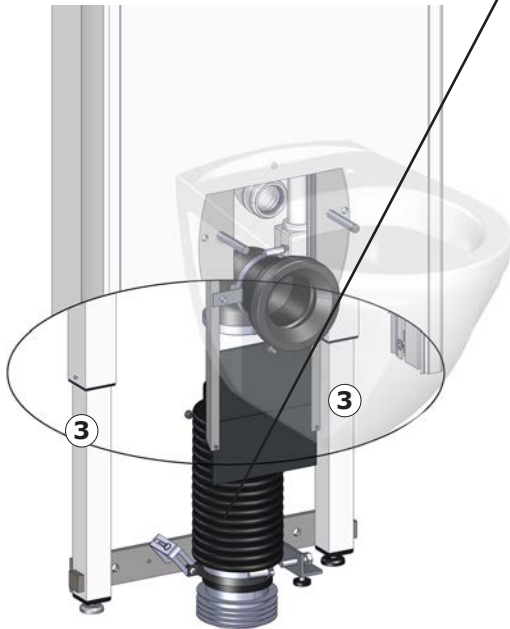
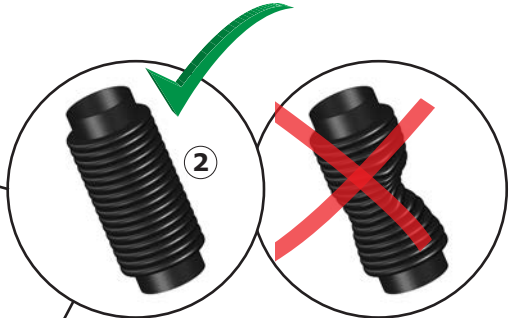
**Disconnect mains before removing the cover.**

Remove the lower cover plate ① (can be clicked off).

en

Reach up under the front cover to unplug the mains plug ① from the control box inside the cabinet.

Ensure that the flexible outlet hose ② can move freely when the toilet is height adjusted. The flexible outlet must be symmetrical when stretched out.

Lubricate the inner square tubes ③ on all 4 sides as far as can be reached.

Reconnect the mains plug.

Take the toilet to its lowest position, and then run it to the top position again to distribute the lubricant.

Reattach the lower cover plate.

en

# Technical data

Max user weight	200 kg / 440 lbs	
Max load	235 kg / 518 lbs	
Product weight	R8212, R8222 R8232, R8242	54,7 kg/ 120,59 lbs
	R8312, R8322 R8332, R8342	60,7 kg / 133,82 lbs

# Trouble-shooting

Symptom	Cause	Action
Operating buttons don't work.	Fault in power supply.	Check that all wires and plugs are intact and correctly connected.
The product runs up and down unevenly.	The sliding brackets are not properly lubricated.	Oil the sliding brackets as described under "Maintenance".

Contact Pressalit A/S or your local dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

# Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Steel	Steel can be recycled by being melted down and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.
Electronics	The electronic components should be collected separately and returned to the designated recycling service.

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